ONCE WEEKLY

MEPTIDE
(TIRZEPATIDE)

INSTRUCTIONS MANUAL

Quick Pen

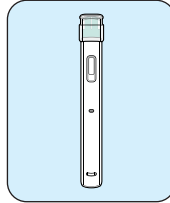
PRE-FILLED PEN AUTOINJECTOR

Wash your hands with soap and Water

Step 1 Check the pen

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- 1 Check the pen label to make sure you have the right medicine and dose, and that it is not expired.
- 2 Make sure the pen is not damaged and is locked.
- 3 Make sure the medicine:
 - is not frozen
 - is colorless to slightly yellow
 - is not cloudy
 - does not have particles

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
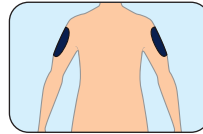
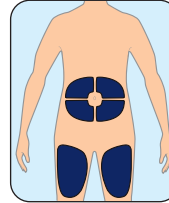
- 1 MEPTIDE can be injected under the skin (subcutaneously) of your stomach area (abdomen) or thighs, and with the help of someone else, in the back of the upper arm.
- 2 Change (rotate) your injection site each week.
- 3 You may use the same area of your body but be sure to choose a different injection site in that area.

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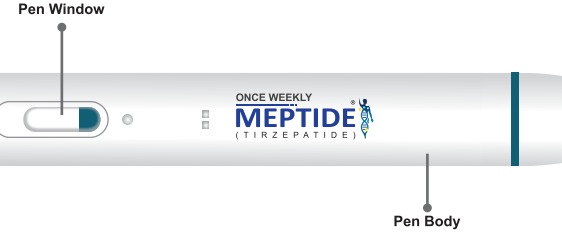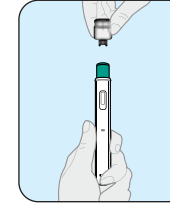
- 1 Pull the pen cap straight off your pen.
- 2 The needle is hidden, so you do not have to see or handle it.
- 3 Be careful not to push down on the needle cover to avoid needle stick injuries.

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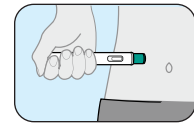
- 1 Push the pen (needle cover sleeve) firmly against your skin and keep applying pressure.
- 2 Listen for click sound during the injection and hold for approximately 10-15 seconds.
- 3 You will know your injection is complete when the plunger rod entirely covers the pen window.
- 4 After using the injection, reattach the pen cap.
- 5 Dispose of the used pen.

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Important Instructions for Use

- Change (rotate) your injection site within the area you choose for each dose to reduce your risk of getting lipodystrophy (pits in skin or thickened skin) and localized cutaneous amyloidosis (skin with lumps) at the injection sites.
- Do not inject where the skin is tender, bruised, scaly or hard, or into scars or damaged skin.
- If you see blood after you take the needle out of your skin, press the injection site lightly. Do not rub the area.
- Do not freeze (if frozen, do not use the pen).
- Do not inject where the skin has pits, is thickened, or has lumps.

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